




# Panbio™

## COVID-19 Antigen Self-Test

A fast, easy, reliable test that provides results in 15 minutes at home. Available in one, four, 10 and 20 test sizes to meet individual and family needs.

### Self-test detects active COVID-19 infection

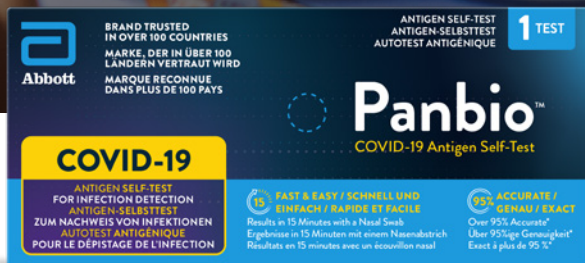

Over 95% accurate\*



Simple test procedure using a nasal swab inserted 2 cm into the nose



Results in 15 minutes. No need to wait for a healthcare provider



No age restriction. Children under 14 years old should be supported by an adult



Clinically proven in individuals with and without symptoms




Illustrated, step-by-step instructions


# SIMPLE PROCEDURE WITH PREFILLED BUFFER TUBE AND COMFORTABLE NASAL SWAB

Read the full Instructions for Use that come in the product kit box before testing.\*\*


## 1 PREPARE MATERIALS

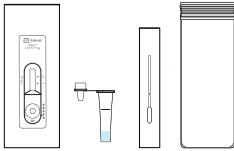


## 2 OPEN TUBE

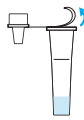

## 3 SWAB NOSTRILS

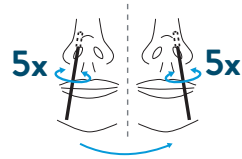

## 4 SWIRL SAMPLE

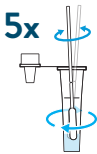

## 5 BREAK SWAB




## 6 ADD 5 DROPS



## 7 WAIT 15 MINUTES



## 8 READ RESULT

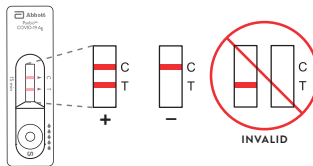

# FREQUENTLY ASKED QUESTIONS

## What is an antigen?

Antigens are proteins that are part of the COVID-19 virus. If the test detects antigens in your nasal sample, it means that an active COVID-19 virus is present in your body.

## How accurate is the test?

In clinical evaluations with self-test users, Panbio™ COVID-19 Antigen Self-Test correctly identified 100% of negative samples and 95.2% of positive samples.

## Can this test be used on children?


The test has no age restrictions. Children under 14 years old should be supported by an adult.

## What does it mean if I have a positive result?

A positive test result means that proteins of the virus that causes COVID-19 have been found in your nasal swab sample. It is likely that you will need to perform self-isolation at home to prevent the spread of COVID-19. Follow local guidelines for social distancing to limit the spread of the virus, and contact your doctor or local health department immediately.

## What does it mean if I have a negative result?

A negative test result means that it is unlikely that you have COVID-19 at the time of testing. The test did not detect any antigens in your nasal swab sample. If symptoms continue, you should repeat the test after 1–2 days because the coronavirus may not be detectable in the very early phases of infection. You are also advised to continue following local guidelines for self-isolation and consult your doctor.

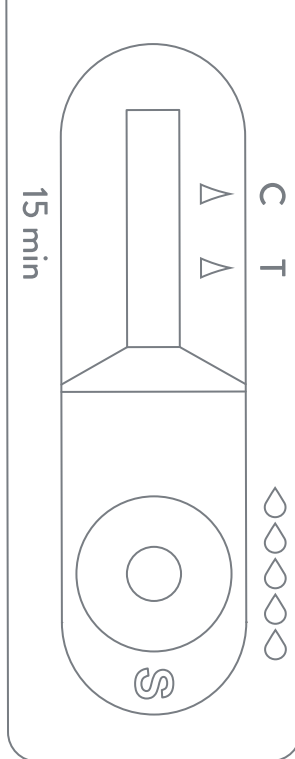

## Does this test detect the new COVID-19 variants?

Abbott is intently monitoring for new mutations of COVID-19 so we can ensure our tests can detect them, as we do with many viruses. We have conducted a thorough analysis of variants that we've been able to study,<sup>1</sup> and we have shown that our tests remain effective at identifying these strains.

## Who can help me if I have a product issue or question?

Technical Support is available by calling the phone numbers listed in the Instructions for Use included in the kit.

FOR MORE INFORMATION,  
VISIT **SELFTEST.ABBOTT**

\*In clinical evaluations with self-test users, the Panbio™ COVID-19 Antigen Self-Test correctly identified 100% of SARS-CoV-2 negative samples and 95.2% of SARS-CoV-2 positive samples.

\*\*These instructions are the test procedure to complete a single test. Kit component quantities may vary.

Product not available in all countries. Available to consumers in select markets.

1. Abbott. Technical Brief on Variants.

© 2022 Abbott. All rights reserved. All trademarks referenced are trademarks of either the Abbott group of companies or their respective owners. Any photos displayed are for illustrative purposes only. Any person depicted in such photos is a model. COL-11665-02 04/22

