



Abbott

HELP STOP THE SPREAD OF COVID-19 OVER-THE-COUNTER RAPID SELF-TEST TO DETECT ACTIVE COVID-19 INFECTION

Panbio™ COVID-19
Antigen Self-Test
detects infection in
asymptomatic and
symptomatic people.



NO AGE RESTRICTIONS

Approved for use on children under 14 years old when supported by an adult.

BRAND TRUSTED IN OVER 100 COUNTRIES



EXCELLENT NASAL SWAB PERFORMANCE**

Nasal swab performance compared to nasal PCR when performed by a trained healthcare professional



PCR = polymerase chain reaction

HIGH-PERFORMANCE SELF-TEST DATA*


The test also showed strong performance when done by self-test users.




EASY PROCEDURE WITH PREFILLED BUFFER TUBE

See Instructions for Use for complete procedure.


1 PREPARE MATERIALS

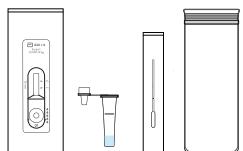


2 OPEN TUBE

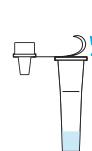

3 SWAB NOSTRILS

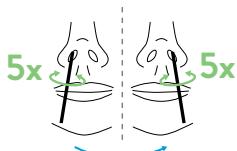

4 SWIRL SAMPLE

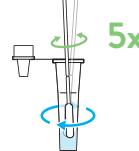

5 BREAK SWAB




6 ADD 5 DROPS



7 WAIT 15 MINUTES



8 READ RESULT

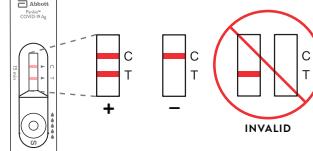

INDIVIDUALLY PACKED WITH INSTRUCTIONS FOR SELF-TESTING BY CONSUMERS

CATALOG NUMBERS

- 41FK51 (1 test)
- 41FK91 (4 tests)
- 41FK71 (10 tests)
- 41FK81 (20 tests)

KIT CONTENTS

• Test device	• Swab
• Instructions for Use	• Bag
• Buffer tube	• Tube rack (10 and 20 test kits)



CONTACT YOUR LOCAL ABBOTT REPRESENTATIVE TODAY
OR VISIT SELFTEST.ABBOTT

*In clinical evaluations with self-test users, the Panbio™ COVID-19 Antigen Self-Test correctly identified 100% of SARS-CoV-2 negative samples and 95.2% of SARS-CoV-2 positive samples.

**In medical diagnosis, sensitivity is the ability of a test to correctly identify those with the disease. Specificity is the ability of the test to correctly identify those without the disease. See product Instructions for Use for additional performance information.

Product not available in all countries. Available to consumers in select markets.

© 2022 Abbott. All rights reserved. All trademarks referenced are trademarks of either the Abbott group of companies or their respective owners. Any photos displayed are for illustrative purposes only. COL-11684-01 04/22

